

PRODUCT CATALOGUE



Our Concept, Your Smile Dental Prothesis Cleaning System





The C2S Pro cleaning system was developed to improve dental care in healthcare facilities. It significantly lowers bacteria and fungi (candidiasis) levels, reducing the occurrence of some systemic diseases (upper respiratory disease). Regular care of dental prostheses improves the general health of residents. In addition, daily use of the C2S Pro reduces referrals to specialists (dentists, physicians and prosthodontists).

Developed for daily cleaning (2 minutes a day) of all types of removable dental prostheses, its cleaning system uses dynamic brushing with special alloy bristles to help extend the life dentures and prevent the formation of plaque and tartar.



Reference: 2CPRO

- Dimensions: L. 20 x W. 17.5 x H. 19.5 cm
- Cleaning operation: magnets rotating at 1800 rpm
- Controls: start/stop button and two high / low buttons to change the timing
- Timing: 5 min / 10 min / 15 min / 20 min, automatic system shutdown
- Certification: CE and medical device
- Indicators: LED screen
- Power supply: separate CE-certified transformer, input: 100–240 V / output:12 V
- Warranty: 2 years

The C2S Pro cleaning system can be used in three different protocols:

- **DAILY** (dirty prostheses): 2 minutes of cleaning with water and one C2S Tab effervescent sodium bicarbonate tablet.
- **INTENSIVE** (prostheses with plaque accumulation): 5 minutes of cleaning with C2S Clean solution diluted 1:3.
- **MEDICATED** (presence of fungus): 10 minutes of cleaning with pure chlorhexidine (in consultation with coordinating physician).



The C2S Pro cleaning system includes a "cleaning box".

It is personalised and replaceable (after 12 months) and provides optimal hygiene, improving user comfort while preventing cross-contamination.



Reference: 2CBOLXX

- Dimensions: Ø 11 x H. 9,5 cm
- Cleaning operation: dynamic cleaning through rotational magnetic stimulation of bristles
- Packing: 2 cleaning boxes
- Available colours: blue, purple, orange, yellow or green
- Reporting: name labels
- Certification: CE and medical device



200 effervescent sodium bicarbonate tablets for cleaning dental prostheses with the C2S medical device

Designed by C2S and manufactured in France in partnership with ADP Laboratoire, C2S Tab clean deeply in just 2 minutes. They are used daily with the C2S Pro cleaning system. Their active ingredients — sodium bicarbonate and cider vinegar — act as antibacterial and descaling agents. The thyme-mint fragrance leaves breath clean and fresh.



Reference: 2CTAB

COMPOSITION:

- Sodium bicarbonate
- Cider vinegar
- Thyme-mint fragrance
- Magnesium stearate
- Microcrystalline cellulose
- Blue colouring (E133)

Using C2S Tab tablets with the C2S Pro medical device:

- 1 Place one tablet with the dental prosthesis in the individual "cleaning box".
- 2 Switch on the C2S Pro for at least 2 minutes.
- 3 Remove and rinse the dental prosthesis.
- 4 Rinse the cleaning box.



Descales, loosens tartar and deeply cleans all types of removable and screw-retained dental prostheses.



Reference: 2CCLEAN

- Weight: 1.4 kg
- Volume: 1 L
- Composition: 15 30% inorganic acid (phosphoric acid)
- Does not change prosthetic structure
- Specially designed for use with the C2S Pro cleaning system

Using C2S Clean with the C2S Pro medical device:

- 1 Place 50 mL of C2S Clean solution and 100 mL of water into the cleaning box (1:3 dilution).
- 2 Switch on the C2S Pro for at least 10 minutes.
- 3 Remove and rinse the dental prosthesis thoroughly.
- 4 Rinse the cleaning box thoroughly.



The C2S Bat battery makes the C2S Pro mobile. The long battery life allows daily use of the C2S Pro cleaning system for a week without recharging (based on daily use of about 1 hour, or 30 two-minute cleanings).



Reference: 2CPROBAT

- Power supply: separate CE-certified Transformer, input: 19 V /output: 19.5 V
- Certification: CE
- Colour: white
- Indicators: LED
- Battery life: 7 hours of operation

+ DENTAL CARE TROLLEY

The dental care trolley is designed to optimize the efficiency and quality of all dental care in health facilities.

It can be equipped with 5 drawers for the storage of prosthese cleaning boxes. Thanks to its standard lateral rails which are compatible with many available accessories, this trolley is easily customizable.

Reference: 2C0100R

- Dim.: L. 61.5 x W. 46.8 x H. 98.8 cm
- Natural anodised structure
- ABS side wall, coloured body
- Work surface with rounded raised edges made of thermoformed plastic
- Rolling shutter and key lock

- 4 swivelling wheels Ø 100 mm, inc. 1 with brake
- 1 handle
- 2 standard rails for accessories
- Colour: orange
 - Delivered without equipment



The accessories for this trolley can be subject to change so please contact us.





Samarit Medical AG Gewerbestrasse 12 CH-8132 Egg, Schweiz Tel.: +41 44 918 10 11 Fax: +41 44 918 24 39 e-mail: info@samarit.com - www.samarit.com

www.samarit.com