BEACH-CHAIR device

Instructions for Use (EN)
Gebrauchsanweisung (DE)
Mode d'emploi (FR)
Manuale d'uso (IT)
Instrucciones de uso (ES)

For other languages please look for more translations on our website.

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MD

product name & ID:

see Table 1: Beach-Chair models and UDI-DI, page 3

class I MDR 2017/745

GMDN code 65180

EMDN code V0804

Manufacturer

Support Center, Headquarters

Samarit Medical AG, Switzerland
Grundstrasse 14, CH-8126 Zumikon

Samarit Medical AG, Switzerland
Gewerbstrasse 12, CH-8132 Egg
Tel.: +41 44 918 10 11
info@samarit.com - www.samarit.com
SRN **CH-MF-000016280**

EU REP

Authorized Representative

Samarit Medical Produkt GmbH
Im See 19, D- 79790 Küssaberg, Germany
Tel.: +41 44 918 10 11
info@samarit.com - www.samarit.com
SRN **DE-AR-000014694**

Incident and Adverse Event:

For any incident or event, please contact immediately:
An Adverse Event Report form is systematically provided to the customer through the company website or per email.

incident@samarit.com
+41 44 918 10 11

Symbols for Cautions and Warnings

Symbol	Symbol explanation	Symbol	Symbol explanation
	Caution related to important information potentially leading to misuse, use error, malfunction or damage of the product		User or patient information: notice related to useful information and recommendation for an efficient use of the system.
	Manufacturer address		Indicates the item is a medical device
	Consult the instructions for use		In conformance with the EU Medical Device Regulation
	European Representative		Unique device identifier – Device Identifier
	Unique device identifier – Device Identifier		Lot Number
	Reference / Order Number		

Copyright statement

© 2016-2025 All rights reserved by Samarit Medical AG, Switzerland. Products are subject to change without prior notice. Samarit Medical AG, Switzerland provides the information in this booklet to be read by anyone, but retains the copyright on all text and graphics.
Beach-Chair is a registered trademark of Samarit Medical AG, Switzerland.

Table of Content

Table of Beach-Chair models	3
(EN) Users Guide	4
(DE) Gebrauchsanweisung	10
(FR) Guide de l'utilisateur	16
(IT) Manuale d'uso	22
(ES) Manual de instrucciones	28
LIMITED WARRANTY	34

Table of Beach-Chair models

Product	REF	UDI UDI-DI
Beach-Chair device		
2nd generation • Ergonomic positioning cushion (head, neck and back) • Incl. 5 sets of consumables (Set contents: 1x Forhead cushion, 1 x Chin cushion, 2 x Thorax cushion, 1 x Wing cushion) • Instruction for use • Video (YouTube) 	420.2002	0764 014 4461249
Beach-Chair consumables		
(set of 10) Set contents: 1x Forhead cushion, 1 x Chin cushion, 2 x Thorax cushion, 1 x Wing cushion	420.0010 B	0764 014 4460556

Table 1: Beach-Chair models and UDI-DI

(EN) Users Guide

For complete and detailed instructions, please consult the Samarit Beach-Chair training videos on the website or with QR code on page 3.

CAUTION: All surfaces that come into contact with the patient must be germ-free/disinfected.

CAUTION: There must be no holes, tears, cuts or other damage to the surfaces of the cushions and pads on the side facing the patient, as otherwise sufficient sterile cleaning cannot be guaranteed.


CAUTION: Check the BC device for cleanliness, integrity and completeness. In particular, the straps and fastening bands (Velcro fasteners) must be undamaged and not worn so that the tensile forces caused by the patient's weight can be safely absorbed.

Preparation


Align the operating table (back and head section) flat, mount the rods for the head section extension at the bottom of the head section.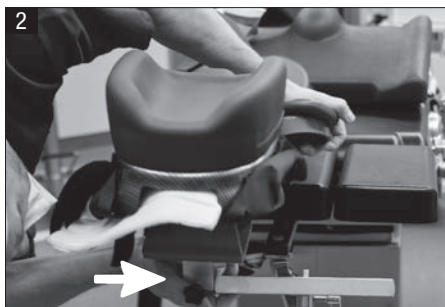

Place the BC device on the head/shoulder section of the operating table and slide the clamp of the head section holder of the BC device onto the operating table rail and clamp it tight.

Ensure that the BC device lies on the center section of the operating table head section

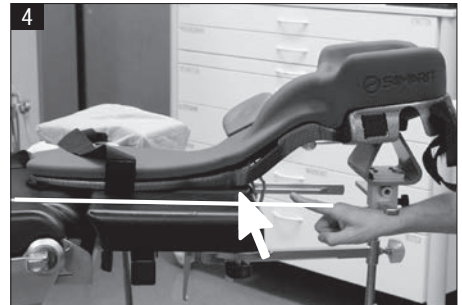
and that the lower section of the BC device lies flat on the operating table. The end of the flat back part of the BC device is positioned flush with the end of the operating table head cushion.

Clamp the brackets securely in this position.

Safety strap is fixed on upper thorax band holder.

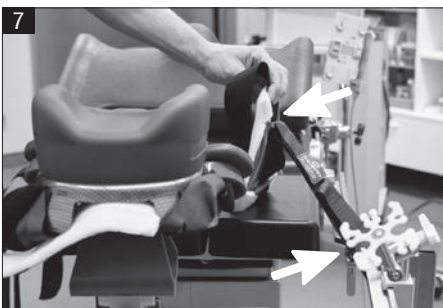Secure the safety strap of the BC device to the operating table rail on the side facing away from the surgeon using the Velcro safety strap.

Position the anti-slip seat cushion on the centre section of the operating table so that the white area at the bottom of the cushion is on the pivot point of the operating table and the cushion is not trapped when the operating table is bent

The leg cushion can be positioned next to it.

Either cover the operating table with a small underlay between the non-slip seat cushion and the BC device.

or cover the operating table including the seat cushion and BC device with a large underlay.

On the side facing the surgeon, the shoulder section can be removed from the operating table if required. The operating table and the BC device are thus prepared for positioning the patient. Transfer the patient to the prepared operating table, adjust the head/shoulder height, tilt of the upper body, buttocks and knee bend according to the planned operation.

Secure the patient's chin and forehead to the head section of the BC device using the disposable padding and Velcro straps.

After setting the desired operating table position, check the patient positioning (correct position of the seat cushion, not jammed).

Set the desired operating table positions and check the positioning of the patient.

Check leg/hip straps and tighten if necessary. Check the chin and headbands.

Removing the shoulder section from the operating table on the surgeon's side provides optimal access for the surgeon. The patient is then ready for the operation.

CLEANING

The Beach-Chair device can be cleaned best with “Incidin® OxyWipe” by ECOLAB or any comparable hospital disinfectant (**no chlorine, no concentrate, no bleach**).

Do not dispose!

Safety instructions

The user is responsible for reading the User's Guide accompanying every product delivered (in original box). The manufacturer assumes no liability for improper use.

For more instruction consult the training video clip at www.samarit.com

The user must ensure that the product is in perfect working condition to comply with hospital safety regulations.

A Beach-Chair device that does not meet the required requirements must be removed from service immediately.

For repairs, call our service number or contact your nearest distributor. Repairs should only be carried out by authorized personnel.

Intended use

The Beach-chair device including accessories is a surgical aid and is used to position the patient safely and stably on the operating table during anaesthesia despite sagging muscles, so that the procedure can be performed efficiently, and the patient does not slip on the table during the operation. The splint also enables safe and rapid intubation for anaesthesia.

Intended medical indication and contraindication

- Shoulder surgery
- Shoulder prosthesis
- Upper arm and collarbone
- Osteosynthesis
- Multiple use: Facial surgery / carotid artery / struma

No contraindications are known to date.

Intended patient population and user profile

Patients regardless of race and gender,
from approx. 15 years / from approx. 50kg

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LIMITED WARRANTY

by SAMARIT MEDICAL AG SWITZERLAND

SAMARIT MEDICAL AG Switzerland warrants your product to be free from defects in material and workmanship for a period of 24 months from the original date of purchase. If you discover a defect in a product covered by this warranty, we will repair at our option using new or refurbished components, or if repair is not possible, replace the item.

Beach-Chair device

24 months warranty

EXCLUSIONS

This warranty covers defects in manufacturing discovered while using the product as recommended by the manufacturer. The warranty does not cover loss or theft, nor does coverage extend to damage caused by misuse, abuse, unauthorized modification, improper storage conditions, lightening, or natural disasters. The warranty does not cover parts that are subject to normal wear and tear replacement requirements, such as.

LIMITS OF LIABILITY

Should the product(s) fail, your sole recourse shall be repair or replacement, as described in the preceding paragraphs. We will not be held liable to you or any other party for any damages that result from the failure of this product. Damages excluded include, but are not limited to the following: lost profits, lost savings, lost data, damage to other equipment, and incidental or consequential damages arising from the use, or inability to use this product. In no case will SAMARIT MEDICAL AG be liable for more than the amount of your purchase price, not to exceed the current list price of the product, and excluding tax, shipping and handling charges.

SAMARIT MEDICAL AG disclaims any other warranties, express or implied.

By installing or using the product, the user accepts all terms described herein.

HOW TO OBTAIN SERVICE UNDER THIS WARRANTY

Copy of invoice with serial no., pictures of product damage send to info@samarit.com.

The cost of shipping to the manufacturer, or authorized repair center or payment of any customs clearance fees or duties are the responsibility of the user.

Return shipping costs to be paid for by SAMARIT MEDICAL AG, only with written acceptance from Samarit.

Samarit Medical AG
Gewerbestrasse 12
CH-8132 Egg, Switzerland