Instructions for Use (EN)
Gebrauchsanweisung (DE)
Mode d'emploi (FR)
Manuale d'uso (IT)
Instrucciones de uso (ES)

For other languages please look for more translations on our website.

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MD

product name & ID:

see Table 1: Glideboard models and UDI-DI, page 3

class I MDR 2017/745

GMDN code 15-693

EMDN code V080504

Manufacturer

Support Center, Headquarters

Samarit Medical AG, Switzerland

Gewerbestrasse 12, CH-8132 Egg

Tel.: +41 44 918 10 11

info@samarit.com - www.samarit.com

SRN **CH-MF-000016280**

EU REP

European Representative

Samarit Medical Produkt GmbH

Im See 19, D- 79790 Küssaberg, Germany

Tel.: +41 44 918 10 11

info@samarit.com - www.samarit.com

SRN **DE-AR-000014694**

Incident and Adverse Event:

For any incident or event, please contact immediately:

An Adverse Event Report form is systematically provided to the customer through the company website or per email.

incident@samarit.com

+41 44 918 10 11

Symbols for Cautions and Warnings

Symbol	Symbol explanation	Symbol	Symbol explanation
	Caution related to important information potentially leading to misuse, use error, malfunction or damage of the product		User or patient information: notice related to useful information and recommendation for an efficient use of the system.
	Manufacturer address		Indicates the item is a medical device
	Consult the instructions for use		In conformance with the EU Medical Device Regulation
	European Representative		Device Identifier
	Unique device identifier – Device Identifier		Lot Number
	Reference / Order Number		

Copyright statement

© 2016–2026 All rights reserved by Samarit Medical AG, Switzerland. Products are subject to change without prior notice. Samarit Medical AG, Switzerland provides the information in this booklet to be read by anyone, but retains the copyright on all text and graphics. Glideboard is a registered trademark of Samarit Medical AG, Switzerland.

Table of Content

Table of Glideboard models	3
(EN) Users Guide	4
(DE) Gebrauchsanweisung	6
(FR) Guide de l'utilisateur	8
(IT) Manuale d'uso	10
(ES) Manual de instrucciones	12
LIMITED WARRANTY	14

Table of Glideboard models

Product	REF	UDI UDI-DI	SWL*	Überbrückbarer Abstand
Glideboard The Biggest (100 x 34 x 1 cm), 2,9 kg	476.0102	7640144460921	220 kg	< 20 cm
Glideboard The Great (80 x 33 x 1 cm), 2,6 kg	476.0080	7640144460938	220 kg	< 20 cm
Glideboard Curved (68 x 23 x 1 cm), 1,2 kg	476.0012	7640144460945	220 kg	< 20 cm
Glideboard Selfglide (50 x 21 x 1 cm), 0,8 kg	476.0050	7640144460952	180 kg	< 20 cm
Glideboard Tetraglide (63 x 23 x 1 cm), 1,1 kg	476.0863	7640144460969	220 kg	< 20 cm
Sit- On Monofil Sliding-Towel (30 x 45 x 0.1 cm), 0,05 kg	476.0200	7640144460976	–	–

Table 1: Glideboard models and UDI-DI

SWL* = Safe Weight limits

(EN) Users Guide

The Samarit Glideboard is a lightweight transfer board made of High Density Polyethylene (HPDE). The special shape is protected by a utility model (design patent). The material is extremely slippery on the upper side. Two anti-slip stickers are attached to the two ends on the underside and there is a practical carrying handle (cut-out) in the centre of the board.

For complete and detailed instructions, you or your caregiver can consult the Samarit Glideboard video on the website or with the QR code.

The patient or carer pushes one end of the Glideboard under the buttocks, which rest on the seat.

The other end must rest securely on the opposite level with the anti-slip surface on which the patient wants to slide.

The patient then glides over the Glideboard themselves or with help. A special slide sheet can facilitate this process (sit-on sheet).

Textile clothing should preferably be used. Materials such as leather can lead to restrictions (increased slip resistance).

As soon as the patient has been transferred to the bed or another level either independently or with support, the Glideboard can be removed.

CLEANING

The Samarit Glideboard is best cleaned with warm soapy water and wiped dry with a cloth. It can also be cleaned with 'Incidin® Oxyqipe' from ECOLAB or another comparable medical disinfectant (**no chlorine, no concentrate, no bleach**). The anti-slip adhesives can be kept dust-free with a damp cloth and thus guarantee the best adhesion to the support surfaces.

Do not dispose!

Safety instructions

The user is responsible for reading the instructions for use supplied with each product in its original packaging. The manufacturer accepts no liability for improper use. Further instructions can be found in the training video at www.samarit.com

The user must ensure that the product is in perfect condition in order to fulfil the hospital's safety regulations.

A Glideboard that does not fulfil the necessary requirements must be taken out of service immediately

The patient and carer must ensure that the anti-slip stickers are undamaged on the underside. These must be fully and securely in place on both levels, before the transfer is carried out.

If you are unsure how to use the Glideboard or feel unstable on it, please consult a medical professional.

Maintenance: Please check monthly to ensure the anti-slip pads are still securely in place. If necessary, new pads can be ordered directly from us using item no. 470.0090.

Useful life: 10 years

Intended use

The Glideboard is a simple aid for transferring a patient with restricted mobility from the centre of the bed or edge of the bed to a wheelchair, car seat or toilet. The patient does not have to be able to stand up, but can change position sitting up independently or with the support of a carer.

Intended medical indication and contraindication

The Glideboard is used to transfer from one level to another, for example: bed to wheelchair, chair to wheelchair, wheelchair to car seat.

No contraindication or known non-applicability.

Intended patient population and user profile

All patients with a patient weight of up to 220 kg (depending on model, see table on page 3)

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LIMITED WARRANTY

by SAMARIT MEDICAL AG SWITZERLAND

SAMARIT MEDICAL AG Switzerland warrants your product to be free from defects in material and workmanship for a period of 2 years from the original date of purchase. If you discover a defect in a product covered by this warranty, we will repair at our option using new or refurbished components, or if repair is not possible, replace the item.

Glideboard

2-year warranty

EXCLUSIONS

This warranty covers defects in manufacturing discovered while using the product as recommended by the manufacturer. The warranty does not cover loss or theft, nor does coverage extend to damage caused by misuse, abuse, unauthorized modification, improper storage conditions, lightening, or natural disasters. The warranty does not cover parts that are subject to normal wear and tear replacement requirements, such as.

LIMITS OF LIABILITY

Should the product(s) fail, your sole recourse shall be repair or replacement, as described in the preceding paragraphs. We will not be held liable to you or any other party for any damages that result from the failure of this product. Damages excluded include, but are not limited to the following: lost profits, lost savings, lost data, damage to other equipment, and incidental or consequential damages arising from the use, or inability to use this product. In no case will SAMARIT MEDICAL AG be liable for more than the amount of your purchase price, not to exceed the current list price of the product, and excluding tax, shipping and handling charges.

SAMARIT MEDICAL AG disclaims any other warranties, express or implied.

By installing or using the product, the user accepts all terms described herein.

HOW TO OBTAIN SERVICE UNDER THIS WARRANTY

Copy of invoice with serial no., pictures of product damage send to info@samarit.com.

The cost of shipping to the manufacturer, or authorized repair center or payment of any customs clearance fees or duties are the responsibility of the user.

Return shipping costs to be paid for by SAMARIT MEDICAL AG, only with written acceptance from Samarit.

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Local distributor
and translation

Samarit Meditalia S.R.L
Via Trieste 49/D
35035 Mestrino (PD) Italia
info@samarit.it

Distributor local
y traductor

Samarit Mediberica S.L.L
C/ ANTONIO MILLON 26 - BAJO B
29780 NERJA, Málaga, España
info@samarit.es

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von SAMARIT MEDICAL AG, SCHWEIZ